The natural way of Sweden

SISSEL® ACUPRESSUR MAT

- Gebrauchsanweisung
- Mode d'emploi
- Instructions for use
- Gebruiksaanwijzing
- Instrucciones de uso
- Istruzioni per l'uso
- Bruksanvisning
- Instrukcja użytkowania

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General information

Read the following user manual carefully and observe the safety instructions and warnings.

SISSEL® Acupressur Mat is a mat with plastic tips for targeted stimulation of various skin receptors, improving blood circulation and relaxation.

The product may only be used in accordance with these instructions for use.

SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these user instructions.

The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on.

Safety instructions


- ! Check the condition of the product before each use. If damaged, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointed objects.
- ! Use the product only on dry, non-slip and straight surfaces.
- ! Stop using if you find the stimuli very uncomfortable.
- ! Use in children and adolescents, during pregnancy and in case of existing health conditions only under supervision and after medical consultation.
- ! The product is not a toy and should be kept out of the reach of children. Do not allow children to play with it.

Product information

Scope of delivery: 1 x SISSEL® Acupressur Mat (two-piece), 1 x instructions for use

Technical specifications

Dimensions (L x W x H):	40 cm x 60 cm x 1,5 cm, two-piece
Height of Acupressur tips:	approx. 1,5 cm
Weight:	approx. 0,81 kg
Material:	TPE (Thermoplastic Elastomers)

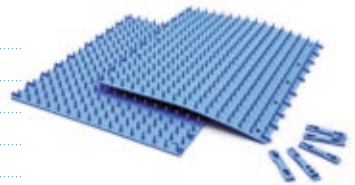Recommendation: From a body weight of 120 kg, it is strongly recommended to wear an outer garment (e.g. a T-shirt).

Note: The Acupressur Mat consists of two partial mats with medium-firm Acupressur tips.

Intended use

- The SISSEL® Acupressur Mat, when used regularly, offers an uncomplicated, individual treatment of various ailments or a simple way to improve well-being and relaxation.
- The SISSEL® Acupressur Mat uses the reactions of the skin, the subcutaneous tissue, the musculature and the metabolic system to the punctiform stimuli of the Acupressur tips.
- In particular, the circulatory and relaxing effects can be targeted on various occasions.

Usage

- Check the product for damage and contamination before each use.
- Assemble the partial mats of the SISSEL® Acupressur Mat on a firm base.
- Place the mat on a flat, soft surface, such as a massage bench, carpet or gym mat. It can also be used on the sofa or in bed.
- Use a pillow or rolled up towel to support the head.
- Beginners start with a simple lying position on the back to get familiar with the feeling on the skin. In the first few minutes, the lying feeling may be uncomfortable. The Acupressur pressure can be regulated by placing a thin towel or putting a T-shirt over it.
- Slowly extend the lying time according to your own feeling, needs and condition. We recommend that you gradually approach the optimum lying time.
- Once you are familiar with the mat, you can achieve targeted effects. A short application time (up to 10 minutes) has a generally stimulating effect, a longer lying time (from approx. 15 minutes) supports relaxation.
- Do not exceed the usage time of max. 40 minutes.
- Advanced users can also use the SISSEL® Acupressur Mat to support relaxation exercises, meditation exercises or targeted to different areas of the body such as arms, legs, or neck.
- Due to the possibility of parting, the mat can be used flexibly as a support for an ordinary pillow for neck treatment or as a seat support.
- Lay down slowly on the mat to avoid a very strong initial stimulus.
- In the beginning, in connection with the use of the SISSEL® Acupressur Mat, redness of the back and imprints of the spikes on the skin may occur. These phenomena are harmless and only temporary.
- Sometimes, after the application, a feeling of slight drowsiness or brief headache or dizziness may occur. This is harmless and represents a natural reaction of the body to the effect of using the SISSEL® Acupressur Mat. Use a thicker underlayer / towel to regulate the pressure.
- Persons suffering from whiplash may initially experience neck pain or dizziness. In this case, the application should be limited to a maximum of 15 minutes every other day. Gradual intensification of the treatment is possible later.
- Always use the product in compliance with the recommendations and instructions described.

Applications

The Acupressur Mat can be used: for muscle tension in the neck, shoulder and back area, to promote blood circulation in the skin and muscles, to stimulate the metabolism, to reduce muscle tension, to support regeneration after sport, for sore muscles, to promote general relaxation, for sleep disorders and stress.

 Warning: Do not sleep on the mat!

 Warning: People with blood clotting disorders, cardiovascular problems, psoriasis, fibromyalgia and pregnancy should consult a doctor before using SISSEL® Acupressur Mat.

 Warning: SISSEL® Acupressur Mat should not be used in cases of epilepsy, major skin damage (e.g. wounds, eczema, sunburn), fever, acute skin infections, fungal infections, thrombosis, fresh fractures or tissue trauma.

- ⚠ Warning: When using, storing and transporting the mat, protect your eyes in particular from accidental contact with the tips.

Care & Hygiene

To clean after use, the SISSEL® Acupressur Mat can be rinsed with hot water and commercial cleaner. Use a soft cloth and no abrasive agents.

- ⚠ Warning: Detergents and disinfectants that can damage the materials of the product must be avoided.
- ⚠ Warning: The application instructions of the detergent and disinfectant manufacturer must be followed.

The use of unsuitable cleaning agents or their incorrect use, will void the warranty and guarantee. The product can be rolled up for storage and stowed to save space. For storage, protect the product from direct sunlight and store in a dark, dry place when not in use.

Disposal & Environment

- Product: The product can be disposed of with household waste.
- Packaging: No special types of packaging are used. The respective valid disposal guidelines must be observed.

Warranty

- The warranty is governed by the statutory provisions.
- Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

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