**SISSEL**

*The natural way*

# SISSEL® BALANCEFIT®

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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## General information

Read the following operating instructions carefully and follow the safety instructions and warnings. The SISSEL® Balancefit® is an air-filled cushion with nubbed profiles for versatile use in balance and coordination training, fitness training and for performing varied gymnastics. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the SISSEL® Balancefit®, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these operating instructions or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

## Safety instructions

- ! Check the integrity of the product before each use. In case of damage, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate proximity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointy objects.
- ! Train only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a rubberized bottom (anti-slip socks). Do not train with shoes and make sure you wear suitable sports clothing.
- ! Choose an exercise program that fits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! Use only accessories recommended by the manufacturer.
- ! Use by children, adolescents and during pregnancy as well as in case of existing health conditions or cognitive impairment only under supervision and after medical consultation.

 The product is not a toy and should be kept out of the reach of children. Do not allow children to play with it.

## Product information

### Scope of delivery

consisting of: 1 x SISSEL® Balancefit®, 1 x Instructions for Use

The SISSEL® Balancefit® is available in red and blue.

## Technical data

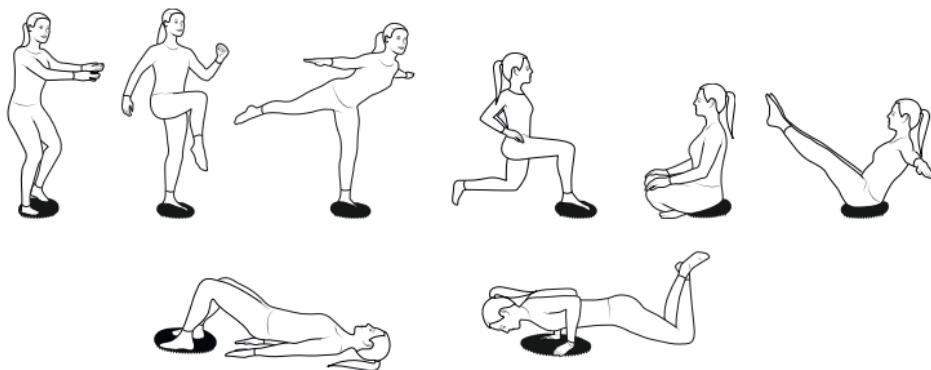
Dimensions (D x H):	ca. 32 x 7 cm
Weight:	ca. 1000 g
Material Casing:	PVC (polyvinyl chloride), phthalate free
Material Filling:	air, variable pressure
Max. user weight:	130 kg

Note: The SISSEL® Balancefit® has two different nub profiles on the top and the bottom.

## Intended use

The SISSEL® Balancefit® serves as a multifunctional sports and training device for the expansion of general fitness training, targeted training in certain sports and, in particular, physiotherapeutic purposes in the prevention, treatment and aftercare of various clinical disorders. Due to the desired instability of the SISSEL® Balancefit®, physiotherapeutic programs for training beginners or inexperienced users up to professional training concepts can be suitably supported. The SISSEL® Balancefit® serves to improve balance, coordination and strengthen muscles. The air-filled plastic body with different nubs to stimulate the soles of the feet and improve sensitivity, is also suitable for exercises while sitting. With a standard ball pump, the air pressure of the SISSEL® Balancefit® can be adjusted and consequently the difficulty of the exercises can be easily increased or decreased.

- ⚠ Therapeutic use should only be done under the supervision or instruction of qualified personnel.
- ⚠ Risk of injury! Only use the product if you can ensure correct performance of the exercise recommendations described. Supervision by qualified personnel is recommended.
- ⚠ Risk of injury! On very slippery or moist surfaces, the SISSEL® Balancefit® can slip away despite its high slip resistance. Therefore, always make sure that the surface is fully covered and dry.

## Indication

Improve body awareness, balance, coordination, responsiveness and depth sensitivity. Strengthening deep muscles, improving muscular coordination, joint stability and promoting psychomotoric skills. Reduction of postural deficits, reduction of the risk of falls and accidents (e.g. geriatrics). Pre- and aftercare for musculoskeletal surgeries, support of rehabilitative measures. Expansion and optimization of the fitness program.

## Contra Indication

Unstable fractures, lack of compliance (e.g., due to anxiety or pain), pronounced balance problems, impairment due to fatigue, medications, alcohol.

## Regulatory requirements

The current product of SISSEL® GmbH complies with the following regulatory requirements:

- Medical Devices Regulation (EU) 2017/745

All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

## Accessory

The product may only be used with the accessories intended for this purpose.

## Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

2. Check the product for integrity and contamination before each use.

3. Use the product as intended.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & Hygiene).

## Care & Hygiene

This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free operation must be ensured. The product can be wiped with commercially available cleaning agents and disinfectants. Use a soft cloth and no abrasive agents. Do not use agents with phenol content or those containing the substance diethylene glycol monoethyl ether.

- ⚠ Only use suitable disinfectants that have been tested for effectiveness to carry out disinfection measures.
- ⚠ Disinfectants that can damage the materials of the product must be avoided.
- ⚠ The application instructions of the disinfectant manufacturer must be followed.

The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee claims.

Storage: Store the SISSEL® Balancefit® in a clean place without direct sunlight. Sunlight can cause discoloration of the surface material. Discoloration does not affect the function.

#### Disposal & Environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

#### Warranty

Note: The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

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SISSEL GmbH