*The natural way*

# SISSEL<sup>®</sup>

## COLD THERAPY

## COMPRESSION

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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## General information

Read the following user manual carefully and observe the safety instructions and warnings. The SISSEL® Cold Therapy Compression is an inflatable cold compress for simultaneous compression and cooling of acute and chronic joint complaints or pain. As a medical device, it is also sold for therapeutic purposes to medical professionals or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the product, e.g. in physiotherapy or rehabilitation treatment, should be carried out by or under the guidance of professional users if possible. Independent use of the product by non-professional users is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

## Safety instructions

- ! Check the intactness of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Leaky, damaged or deflated compresses must not be used.
- ! Low temperatures may cause frost damage to the skin. If necessary, use a thin cotton cloth as an intermediate layer at very low temperatures to prevent skin damage.
- ! Check the temperature before using the compress by gently holding the compress against the inside of the wrist.
- ! Remove the compress immediately if the cold should make you uncomfortable.
- ! Children or persons with cognitive disorders should only use the compress under supervision.
- ! The contents of the compress are not suitable for consumption.
- ! Use only accessories intended by the manufacturer.
- ! The compress and its accessories are not toys and should be kept out of the reach of children.  
Do not allow children to play with it.

## Product information

### Scope of delivery

consisting of: 1 x SISSEL® Cold Therapy Compression – Wrist, 1 x user manual

consisting of: 1 x SISSEL® Cold Therapy Compression – Ankle, 1 x user manual

consisting of: 1 x SISSEL® Cold Therapy Compression – Knee joint / Elbow, 1 x user manual

## Technical data

Dimensions (L x W x H):	Wrist – approx. 22 x 14 x 4 cm Ankle – approx. 26 x 16 x 8 cm Knee joint / Elbow – approx. 33 x 17 x 6 cm
Material: Cover:	Black: 100 % nylon; Blue: Velour (65 % polyester, 35 % cotton)
Velcro fastener:	polyamide
Hand pump / hose:	soft PVC, phthalate-free
Filling:	sodium carboxymethyl cellulose, distilled water, propylene glycol
Weight:	Wrist – approx. 0,46 kg Ankle – approx. 0,50 kg Knee joint / Elbow – approx. 0,68 kg

Note: The inner cold pad of the SISSEL® Cold Therapy Compression can be removed and stored in the refrigerator, freezer or deep freezer until needed. Each variant of the SISSEL® Cold Therapy Compression is equipped with a valve and a hand pump. With their help, the pressure can be individually adjusted at any time. The special gel filling enables long-lasting and safe temperature storage.

## Intended use

The SISSEL® Cold Therapy Compression are used as an aid in local cold compression therapy for acute or chronic joint complaints. The combination of cold and pressure in the form of a joint-encircling bandage achieves a high degree of effectiveness. Cold relieves pain and reduces swelling, while simultaneous compression stabilizes and supports the joint and supports the effect of the cold stimulus. Place the removable cold pads in the refrigerator, freezer or deep freezer (approx. -15 °C to -18 °C). The lower the temperature, the longer the cooling effect of the cold pad. Apply the compress only as long as you find it comfortable – max. 20 minutes. Before reapplying the compress, the treated area should remain uncovered for 20 minutes.

## Application

Place the cold pad in the compress and apply the compress to the joint in question (wrist, ankle, knee joint, elbow). Close the velcro fasteners. Do not apply too tightly or too loosely. For the knee / elbow compress, the joint must be in the joint cutout of the compress. Now open the valve on the compress (set to "ON"). Inflate the compress until you feel a light, stabilising pressure. Use the release valve to release the pressure for readjustment. Do not over-inflate the compress and always make sure that it is correctly seated. Now close the valve on the cuff (set to "OFF") and remove the hose if necessary.

## Removal

Open the valve on the compress (set to "ON"). Release the air from the compress by pressing the release valve or pulling off the hose on the valve. Now loosen the velcro fasteners, remove the compress and press out the remaining air. Note: The SISSEL® Cold Therapy Compression can be applied directly to the skin. At very low temperatures you should use a thin cotton cloth as an intermediate layer to avoid hypothermia. In an emergency, the compress can also be applied directly to clothing. Make sure that the compress fits correctly.

## Indication

Local joint irritation (swelling, bruising, inflammation), sports injuries, pain relief, traumatic and rheumatoid arthritis, near-joint hematomas (bruises), sprains (sprains), dislocations, acute trauma with ligament/joint involvement (meniscus injuries, cruciate ligament stretching/rupture, ligament stretching, patella dislocation), postoperative decongestion and anti-inflammation, early postoperative mobilization. If in doubt, seek medical advice.

## Contraindication

Cold intolerance and cold allergy, Raynaud's syndrome, circulatory disorders, open wounds, peripheral arterial occlusive disease, deep vein thrombosis, impaired skin sensation (e.g. diabetics, heat/cold insensitive persons, children under 3 years of age), unclear findings. If in doubt, seek medical advice.

## Regulatory requirements

The present product of SISSEL® GmbH complies with the following regulatory requirements:

- Regulation (EU) 2017/745 on medical devices.

All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

## Accessories

The product may only be used with the accessories intended for this purpose.

## Usage

1. Check the packaging and the product for any damage upon delivery.  
 Never use a damaged product!
2. Check the product for integrity and contamination before each use.
3. Use the compress as intended.
4. Never place the compress directly on the skin at very cold temperatures. Always check the temperature before applying and use a thin cotton cloth as an intermediate layer if necessary.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & Hygiene).

## Care & Hygiene

This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free function must be ensured. If the compress becomes stained, it can be wiped off with a soft cloth and lukewarm water. Then dry it off and let it air dry. The product can be wiped with commercially available mild disinfectants. Use a soft cloth and no abrasive agents. Do not use any agents containing phenol or those containing the substance diethylene glycol monoethyl ether.

-  Disinfection measures are to be carried out exclusively with suitable disinfectants that have been tested for effectiveness.
-  Disinfectants that can damage the materials of the product are to be avoided.
-  The application instructions of the disinfectant manufacturer must be observed.

The use of unsuitable disinfectants or disinfectants that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee.

## Storage

For storage, protect the product from direct sunlight and store in a clean, dark, dry place when not in use.

## Disposal & Environment

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

## Warranty

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified without the consent of another party or if the user manual has not been followed. Operators of this product must observe the regulatory requirements for medical products applicable at the place of use.

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