The natural way

SISSEL® DORSABACK & DORSABACK-CAR

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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General information

Read the following instructions for use carefully and observe the safety instructions and warnings. The SISSEL® DorsaBack, tried and tested in the USA for many years, allows you to sit in a healthy, upright position even in deeper or softer armchairs thanks to the anatomically shaped wooden core. The seat and backrest made of natural, ergonomically shaped, multi-glued hardwood are joined together and upholstered with a comfortable cover. With its anatomically shaped wooden core, the SISSEL® DorsaBack-Car promotes comfortable, relaxed and back-friendly sitting on long car journeys. The product may only be used in accordance with these instructions for use. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these user instructions. The instructions for use must be kept carefully and within easy reach, and must be enclosed if the product is passed on.

Safety instructions

- ! Check the condition of the product before each use. If damaged, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Use only accessories recommended by the manufacturer.

 The product is not a toy and should be kept out of the reach of children. Do not allow children to play with it.

Product information

Scope of delivery consisting of one of the following products:

1 x SISSEL® DorsaBack, 1 x instruction for use

1 x SISSEL® DorsaBack-Car, 1 x instruction for use

The SISSEL® DorsaBack/DorsaBack-Car is available in black and gray.

Technical data

SISSEL® DorsaBack

Dimensions backrest (L x W x H): approx. 49 x 32,5 x 3,5 cm

Dimensions seat (L x W x H): approx. 39,5 x 43,5 x 3 cm

Material: Inlay: Birch veneered

Filling: Polyurethane foam

Cover: 100 % polyester

Weight: approx. 1,7 kg

SISSEL® DorsaBack-Car

Dimensions (L x W x H): approx. 49 x 36 x 3,5 cm

Material: Inlay: Birch veneered

Filling: Polyurethane foam

Cover: 100 % polyester

Weight: approx. 0,98 kg

Accessory

The product may only be used with the intended accessories. The SISSEL® DorsaBack pad is available separately as a matching, height-adjustable lumbar support.

Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

SISSEL® DorsaBack

2. The SISSEL® DorsaBack fits on any couch, armchair or chair and is easy to transport thanks to its low weight.

SISSEL® DorsaBack-Car

Note: Ensure that the use of the SISSEL® DorsaBack-Car is permitted in accordance with the car seat manufacturer's instructions. Some manufacturers may prohibit the use of additional products such as pads or cushions as this could affect the safety of the seat.

1. The SISSEL® DorsaBack-Car can be easily attached to any car seat using the retaining straps.

Note: Make sure that the SISSEL® DorsaBack-Car is compatible with the specific car seat you are using. Different car seats may have different shapes and sizes, so the overlay should fit correctly to ensure optimum safety.

 Risk of injury! Check the SISSEL® DorsaBack-Car regularly for signs of wear or damage. Replace it if necessary to ensure safety.

 Risk of injury! The SISSEL® DorsaBack-Car cannot be used if side airbags are installed in the seat backrests.

 The SISSEL® DorsaBack-Pad is not suitable for use on a car seat or in combination with SISSEL® DorsaBack-Car on a car seat.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & hygiene).

Care & hygiene

The SISSEL® DorsaBack/DorsaBack-Car can be carefully rubbed with a damp cloth or upholstery foam.

Note: When using upholstery foam, always test for material compatibility in an inconspicuous place beforehand.

 The SISSEL® DorsaBack/DorsaBack-Car cannot be disinfected!

The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee claims.

Disposal & environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

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SISSEL GmbH