*The natural way*

SISSEL®

# HOT-COLD PEARL MASKS & PACKS

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# SISSEL® HOT-COLD PEARL MASKS & PACKS

## General information

Read the following user manual carefully and follow the safety instructions and warnings. The SISSEL Hot-Cold Pearl compresses are reusable cold / warm compresses filled with gel beads for targeted and local cold or heat therapy. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the product or how to avoid them. The therapeutic use of the product, e.g. in physiotherapy or rehabilitation treatment, should be carried out by or under the guidance of professional users if possible. Independent use of the product by non-professional users is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL GmbH accepts no liability for the use of the product outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

## Safety instructions

- ! Check the integrity of the product before each use. If damaged, the function of the product and safety are no longer guaranteed, leaking hot gel can cause skin burns.
- ! Heat the product only in the microwave. Do not heat the compress in a water bath, oven or grill.
- ! Do not leave the microwave unattended when heating the compress. Overheating can destroy the compress and cause skin burns. If the compress puffs up, stop the heating process immediately. After cooling, check that the compress is intact and choose a shorter heating time.
- ⚠ Fire hazard! Make sure that the plate can rotate freely in the microwave!
- ⚠ Burn hazard! Knead the compress well after each heating. This way the heat is evenly distributed in the filling and no heat pockets are created.
- ! Never place the compress directly on the skin! Always use a thin cotton cloth as an in-between layer to avoid damage to the skin (e.g. frostbite, chilblains, burns).
- ! Check the temperature by carefully holding the compress to the inside of the wrist.
- ! Remove the compress immediately if the cold or heat makes you uncomfortable.
- ! Children or persons with cognitive impairments should use the compress only under supervision.

 Burn hazard! Leaking, damaged or deflated compresses must not be used.

! The contents of the compress are not suitable for consumption.

! Use only accessories intended by the manufacturer.

! The compress and its accessories are not toys and must be kept out of the reach of children.

Do not allow children to play with it.

## Product information

### Scope of delivery

consisting of: 1 x SISSEL Hot-Cold Pearl Compress, 1 x Instructions for use

### Technical specifications

Material: Cover: PVC foil

Filling: Gel beads (water, glycerol, sodium polyacrylate)

Dimensions (L x W): SISSEL Hot-Cold Pearl Eye Mask: approx. 23 x 10 cm open,  
with velcro fastener

SISSEL Hot-Cold Pearl Facial Mask: approx. 21 x 18,5 cm open,  
with Velcro fastener and openings for eyes / nose / mouth

SISSEL Hot-Cold Pearl Relief Pack: approx. 27 x 14 cm

SISSEL Hot-Cold Pearl Sports Pack: approx. 17 x 11 cm

### Intended use

The SISSEL Hot-Cold Pearl compresses offer a drug-free, need-based, individual cold or heat therapy. Cold and heat applications use the reactions of the skin, subcutaneous tissue, muscles and deep tissue layers to the respective temperature stimulus. In heat therapy, these are blood circulation-promoting and metabolism-stimulating effects, and in cold therapy, heat withdrawal and pain relief. Thus, the compress is an ideal way to use cold and heat in a targeted therapeutic manner on various occasions.

### Application cold

Place the SISSEL Hot-Cold Pearl compress in the freezer for at least 2 hours (approx. -15 °C to -18 °C). Dry the compress if necessary. Apply the compress to the affected area, using a thin cotton cloth as an intermediate layer if necessary. Only apply the compress for as long as you find it comfortable - max. 20 minutes. Before applying the compress again, the treated area should remain uncovered for 20 minutes.

**Application heat**

Heat the compress only in the microwave.

**Note:** The specified heating times apply to products with an initial temperature of approx. 20 °C (room temperature). Products that have been cooled or frozen must be brought to room temperature before heating.

Heat the SISSEL Hot-Cold Pearl Compress gradually in the microwave. Make absolutely sure that the compress can heat up freely in the microwave and that there are no food residues in the microwave. For microwaves with a turntable, ensure that the plate can rotate freely! Place the compress flat in the microwave. If the compress extends beyond the microwave plate edge, fold it loosely for heating. The actual performance of different microwave brands varies despite the same power specifications. Please refer to the following instructions to reach the desired temperature. Heat the compress according to the following instructions:

<b>Product</b>	<b>Power</b>	<b>Prescribed heating time</b>	<b>Max. total heating time</b>
SISSEL Hot-Cold Pearl Eye Mask	700 W	15 seconds	25 seconds
	1000 W	12 seconds	22 seconds
SISSEL Hot-Cold Pearl Facial Mask	700 W	45 seconds	55 seconds
	1000 W	35 seconds	45 seconds
SISSEL Hot-Cold Pearl Relief Pack	700 W	40 seconds	50 seconds
	1000 W	35 seconds	45 seconds
SISSEL Hot-Cold Pearl Sports Pack	700 W	20 seconds	30 seconds
	1000 W	10 seconds	20 seconds

After the prescribed heating time specified in the table, carefully check the temperature of the compress on the inside of the wrist and knead it thoroughly to distribute the heat evenly. The heating process may be repeated once for a maximum of 10 seconds, provided that the maximum total heating time specified in the table is not exceeded. Once the maximum total heating time has been reached, allow the product to cool completely before reheating.

- Burn hazard!** Do not exceed the maximum total heating time specified in the table!
- Risk of injury!** The heat may be unevenly distributed at first. It is important to knead the compress well after each heating interval to distribute the temperature evenly in the gel.
- Risk of injury!** Never place the hot compress directly on the skin, always use a thin cotton cloth.

**Indication cold**

Local tissue irritation conditions (swellings, bruises), sports injuries, pain relief, traumatic and rheumatoid arthritis, hematomas (bruises), distortions (sprains), local scalds or burns, insect bites, itching, headaches, migraines, nosebleeds, fever, tired and heavy legs.

**Indication heat**

Tension, muscle soreness, myogelosis / hard tension, back pain, tendopathies, muscle and tendon shortening, lumbago, menstrual cramps, myalgias, gastrointestinal or genitourinary tract irritations, hypertension (high blood pressure), general nervousness and restlessness, colds.

**Contraindication cold**

Intolerance to cold and allergy to cold, Raynaud's syndrome, circulatory disorders, acute trauma (accidents, open wounds), peripheral arterial occlusive disease, deep vein thrombosis, impairment of skin sensation (e.g. diabetics, persons insensitive to heat / cold, children under 3 years). If in doubt, seek medical advice.

**Contraindication heat**

Acute inflammation, acute trauma (accidents, open wounds), acute joint inflammation, edema, febrile infections, dermatitis, eczema, deep vein thrombosis, peripheral arterial occlusive disease, hypotension, cardiovascular disease, impaired skin sensation (e.g. diabetics, persons insensitive to heat / cold, children under 3 years). If in doubt, seek medical advice.

**Regulatory requirements**

The present product of SISSEL GmbH complies with the following regulatory requirements:

- Regulation (EU) 2017 / 745 on medical devices.

⚠ All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

**Accessory**

The product may only be used with the accessories intended for this purpose.

**Use**

1. Check the packaging and the product for any damage upon delivery.

⚠ Never use a damaged product.

2. Check the product for integrity and contamination before each use.

3. Never place the compress directly on the skin in very cold or very hot temperatures. Always check the temperature before applying and use a thin cotton cloth as an intermediate layer if necessary.
4. Use the compress as intended.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & hygiene).

### **Care & hygiene**

This product is suitable for reuse. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed, and error-free function must be ensured. In case of soiling, the compress can be wiped off with a soft cloth and lukewarm water. Then leave to dry in the air. If necessary, a mild disinfectant soap can be used for cleaning. Use a soft cloth and no abrasive agents. Do not use agents containing phenol or those containing the substance diethylene glycol monoethyl ether. Sanitizers, abrasive cleaners, glass cleaners or all-purpose cleaners can damage the surface materials of the product and must therefore not be used. The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee.

Storage: Keep out of direct sunlight. Store SISSEL Hot-Cold Pearl compress in a dry, dark, clean place.

### **Disposal & environment**

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

### **Warranty**

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

Operators of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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