The natural way

SISSEL[®] HYDROTEMP[®]

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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General information

Read the following user manual carefully and follow the safety instructions and warnings. The SISSEL® HYDROTEMP® is a reusable moist heat pad for targeted, local and deep-penetrating heat therapy. Available in two different fits, the SISSEL® HYDROTEMP® extracts moisture from the air and releases it as pure, deep penetrating moist heat after heating in the microwave. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the product, e.g. in physiotherapy or rehabilitation treatment, should only be carried out by or under the guidance of professional users in a patient-specific manner. Independent use of the product by non-professional users is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the integrity of the product before each use. If damaged, the function of the product and safety are no longer guaranteed, leaking hot gel can cause skin burns.
- ! Only heat the product in the microwave. Do not heat the cushion in the oven, in a bain-marie or in the grill.
- ! Do not leave the microwave unattended when heating the heating pad. Overheating can destroy the product and cause skin burns. If the product puffs up, stop the heating process immediately. Then check the integrity of the heating pad after it has cooled down and select a shorter heating time.
- ! Shake the heating pad briefly after heating. This distributes the heat evenly and prevents heat pockets from forming (risk of burns).
- ! Check the temperature by carefully holding the heat pad against the inside of the wrist.
- ! Remove the heat pad immediately if you find the heat uncomfortable.
- ! Children or persons with cognitive impairments should only use the heating pad under supervision.
- ! Leaking, damaged or deflated heating pads must not be used. Danger of burns!
- ! The contents of the heating pad are not suitable for consumption.
- ! Only use accessories intended by the manufacturer.
- ! The heating pad and accessories are not toys and must be kept out of the reach of children.
Do not allow children to play with it.

Product information

Scope of delivery

consisting of: 1 x SISSEL® HYDROTEMP®, 1 x instruction manual

The SISSEL® HYDROTEMP® is available in two versions: neck/shoulder (triangular) and back

Technical data

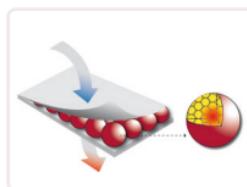
Dimensions (L x W): Neck/shoulder (triangular): approx. 49 x 25.5 cm with two strings

Back: approx. 32.5 x 19 cm with two belts

Material Cover: Polyester, polyurethane

Material Filling: Microspheres of silicon oxide, sodium oxide, aluminium oxide, magnesium oxide, quartz

Note: The SISSEL® HYDROTEMP® has a pleasantly skin-friendly cover. The latex-free filling made of special microspheres stores moisture from the air and releases it again after heating. This creates the deep-acting moist warmth, which adapts perfectly to the contours of the body thanks to the two variants.

Intended use

The SISSEL® HYDROTEMP® offers a drug-free, need-based, deep-acting, individual heat therapy. Heat applications use the reactions of the skin, the subcutis, the musculature and deep tissue layers to the heat stimulus. The moist warmth of the SISSEL® HYDROTEMP® means that the circulation-promoting and metabolism-stimulating effects of the therapy are more intensive than with the usual dry warmth of hot water bottles, heating pads or grain pillows. Heat the SISSEL® HYDROTEMP® gradually in the microwave. Make absolutely sure that the plate with the heating pad can rotate unhindered in the microwave and that there is no food residue in the microwave! Place the pad flat in the microwave. If the heating pad protrudes over the microwave plate rim, fold it loosely for heating. Halfway through the heating time, remove the pad, fold it in the opposite direction and continue heating. The actual performance of different microwave brands varies despite the same power specifications. We therefore recommend that you try to find the optimum heating time.

You can use the following information as a guide:

- With 600 watts or 800 watts
 - Heat for 1 minute and check the temperature on the inner wrist.
 - Knead the pad well beforehand to distribute the heat evenly.
 - If necessary, heat the heat pad further in 20 second steps. Do not exceed a total heating time of 2 minutes!
- The heat storage time is about 30 minutes.

Heating in the microwave enables precise regulation of the temperature. Unintentional „reheating“ of the heat pad is impossible. Place the SISSEL® HYDROTEMP® directly on the bare skin or use a thin cotton cloth as an intermediate layer (slight soaking possible). After use, store the SISSEL® HYDROTEMP® open, not air-tight. This allows the special microspheres to absorb humidity from the environment. Before repeated use, the SISSEL® HYDROTEMP® should rest for 2 hours.

 Risk of injury! Only use the product after carefully checking the warmth. It is important to knead the pad well after each heating interval to distribute the temperature evenly in the product.

Indication

Tension, sore muscles, myogelosis/hard tension, back, neck and shoulder pain, tension headaches, tendopathies, muscle and tendon shortening, lumbago, menstrual cramps, myalgias, irritations of the gastrointestinal tract, hypertension (high blood pressure), general nervousness and restlessness, colds.

Contraindication

Acute inflammation, acute trauma (accidents, open wounds), acute joint inflammation, lack of compliance (anxiety or pain), oedema, febrile infections, dermatitis, deep vein thrombosis, peripheral arterial occlusive disease, hypotension, heart disease, impaired skin sensation (e.g. diabetics, persons insensitive to heat/cold, children under 3 years). If in doubt, seek medical advice.

Regulatory requirements

The SISSEL® HYDROTEMP® from SISSEL® GmbH fulfils the following regulatory requirements:

- Regulation (EU) 2017/745 on medical devices.

All serious incidents occurring in connection with the product must be reported immediately to the manufacturer and the competent authority.

Accessory

The product must only be used with the accessories provided.

Usage

1. Check the packaging and the product for any damage on delivery.

 Never use a damaged product.

2. Check the product for integrity and contamination before each use.
3. Check the temperature of the product after each heating process and before applying it and use a thin cotton cloth as an intermediate layer if necessary.
4. Use the product as intended.

Note: Cleaning measures may be necessary after application (see Maintenance & Hygiene).

Maintenance & Hygiene

This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and fault-free functioning must be ensured. The SISSEL® HYDROTEMP® can be wiped with a moderately damp cloth, do not completely moisten and soak. Do not wash in the washing machine! Then allow to air dry, turning repeatedly. Do not dry on the heating, in the microwave or under direct sunlight.

Storage: Protect from direct sunlight. Store the SISSEL® HYDROTEMP® in a dry, dark, clean place.

Disposal & Environment

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

Warranty

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. Operators of this product must observe the regulatory requirements for medical products applicable at the place of use.

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