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Ergonomic knee support cushion with velcro fastening straps to prevent unintentional slipping.

The ergonomically shaped cushion provides comfortable padding and relief for the knee joints when lying on your side. Unpleasant pressure pains caused by knees lying on top of each other are actively reduced when lying in a lateral position. The soft velcro fastening straps are easy to attach and ensure sufficient stability and minimal movement of the cushion, even during sleep. The skin-friendly velour cover can be washed separately for better hygiene.

How to attach the support cushion

Position the knee support cushion between your knees and attach it to your leg using the attached velcro straps.

Please Note

When attaching the fastening straps, always make sure to apply minimum force in order to ensure optimum blood circulation and prevent high pressure to your skin.

Care

The Cover consists of blended cotton-polyester fabric. It is removable and washable at 40° C. The Core is made of polyurethane foam which can be hand washed using mild detergent. Dry carefully turning several times.

- | easy to attach with 2 velcro straps
- | prevents pressure on knee joints
- | optimal lying position, the pelvis is at a better angle
- | prevents unpleasant sweating
- | knee cushion with a soft velour cover, 75% cotton, 25% polyester
- | dimensions: 17 x 20 cm, 10 cm on the inside

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