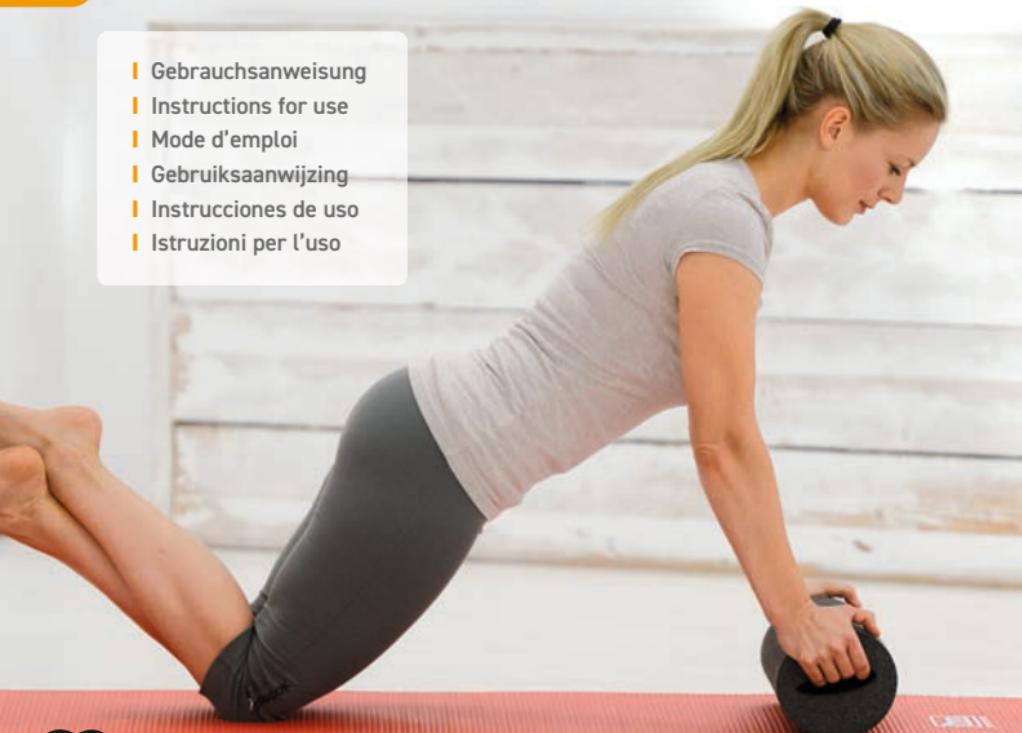
The natural way

SISSEL[®]

MYOFASCIA ROLLER

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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General information

Read the following user manual carefully and follow the safety instructions and warnings. The SISSEL® Myofascia Roller is a universal massage roller for easy self-use and as a multifunctional training tool. Thanks to the handle recesses on the side, the fascia roller enables intensive exercises for regeneration and endurance. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the fascia roll, e.g. in physiotherapy or rehabilitation treatment, should be carried out by or under the guidance of professional users if possible. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories must be used in accordance with these instructions for use or as instructed by appropriately trained professionals. SISSEL® GmbH accepts no liability for the use of the products outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the integrity of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Do not use the product in the close area of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointed objects.
- ! Exercise only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train only barefoot or wear socks with a rubberized bottom (anti-slip socks). Do not train with shoes and make sure you wear suitable sports clothing.
- ! Choose a training program that suits your physical strength and fitness level and avoid too difficult exercises or experiments with increased risk.
- ! Do not use the product with more than one person at the same time.
- ! Use only accessories intended by the manufacturer.
- ! Use in children, adolescents and during pregnancy as well as in case of existing health complaints or cognitive impairments only under supervision and after medical consultation.
- ! The product is not a toy and should be kept out of reach of children. Do not allow children to play with it.

Product information

Scope of delivery

SISSEL® Myofascia Roller

consisting of: 1 x Myofascia Roller, 1 x user manual

The SISSEL® Myofascia Roller is available in black, blue and white-blue (soft version).

Technical data

Dimensions (L x D): approx. 40 x 15 cm

Material: EPP (expanded polypropylene) - recyclable, odorless, dimensionally stable and robust

Max. user weight: 120 kg

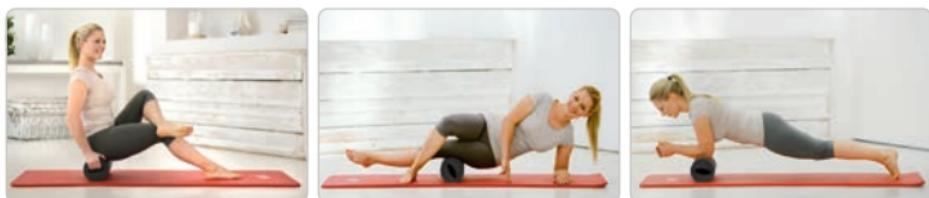
Notice: The Myofascia rollers with the colours blue and black have a higher degree of hardness than the white and blue roller (soft variant). On both sides of the roller are anatomically shaped grip recesses for greater safety during training.

Intended use

The SISSEL® Myofascia Roller is a roller for effective self-massage of stressed muscles as well as for active relaxation of myofascial tissue (fascia) and thus for relaxing and releasing the muscles as well as for stimulating the metabolism. By rolling the extremities and the back with the tight material, fascia and muscles are specifically targeted, loosened as well as increased blood circulation.

The use of the SISSEL® Myofascia Roller has short and long-term effects on the fasciae and the musculature and thus on the entire musculoskeletal system. In the short term, blood circulation and metabolism are stimulated. In the long term, positive effects on joint mobility, the stretching ability of the musculature, posture and everyday movements are achieved.

The SISSEL® Myofascia Roller is suitable for autonomous use at home or in professional use in physiotherapy and rehabilitation for various medical indications. Due to the versatile application possibilities of the SISSEL® Myofascia Roller, targeted exercise programs for inexperienced users up to professional training concepts can be suitably supplemented and supported.

Notice: At the beginning and depending on the degree of tension of the muscles, the effect of using the SISSEL® Myofascia Roller can be experienced as very intense and slightly painful. Adjust the duration and intensity of use to your individual conditions. Use a damping gymnastics mat as a training surface. If you experience persistent pain, interrupt your workout and consult a doctor if necessary.

⚠️ Risk of injury! Only use the product if you can ensure correct performance of the exercise instructions described. Instruction by qualified personnel is recommended.

 Risk of injury! Excessive pressure during application can lead to irritation of veins, nerves, muscles and skin due to biomechanical overload. The individual pain threshold should not be exceeded. Rolling should not be done directly over the spine and joints.

Indication

Movement restrictions, tense, hardened muscles, diffuse pain in the knee, hip, shoulder, neck (consultation with treating physician), pain in the lumbar spine area, sports-related, active regeneration, sports-related injury prophylaxis, therapeutic aid in physiotherapy, osteopathy, sports medicine, supporting aid in yoga and Pilates, supporting aid in measures to improve posture, body feeling, movement sequences and balance.

Contraindication

Acute injuries (open wounds, soft tissue injuries, fractures), acute radiating pain, sensory disturbances, movement disorders (consultation with attending physician), osteoporosis, intervertebral disc problems, varicose veins, lymphedema, circulatory disorders, thrombosis (consultation with attending physician), blood thinning, pregnancy (consultation with attending physician).

Regulatory requirements

The present product of SISSEL® GmbH complies with the following regulatory requirements:

- Regulation (EU) 2017/745 on medical devices.

All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessories

The product may only be used with the accessories intended for this purpose.

Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product!

2. Check the product for intactness and contamination before each use.

3. Use the product as intended.

4. Warm up well before the workout.

5. Exercise according to the instructions for exercise or as instructed by professionals and trainers.

Notice: Cleaning and, if necessary, disinfection measures may be required after use (see Care & Hygiene).

Maintenance & Hygiene

This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free function must be ensured. Clean the SISSEL® Myofascia Roller with a damp cloth and then rub it thoroughly dry. If necessary, the surface of the product can be wipe-disinfected with a suitable disinfectant. Disinfectants containing corrosive substances (e.g. acids, alkalis), scouring agents, glass cleaners or all-purpose cleaners will damage the surface materials of the product and must not be used.

-  Disinfection measures are to be carried out exclusively with suitable disinfectants that have been tested for effectiveness.
-  Disinfectants that can damage the materials of the product must be avoided.
-  The application instructions of the disinfectant manufacturer must be observed.

The use of unsuitable products or products that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee.

Disposal & Environment

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

Warranty

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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