The natural way

SISSEL® SITFIT® & SITFIT® PLUS

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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General information

Read the following operating instructions carefully and follow the safety instructions and warnings. The SISSEL® SITFIT® / SITFIT® Plus is a multifunctional, ergonomic, air-filled seat cushion for active and anatomically correct sitting and a training aid for several exercises while sitting and standing. As a medical device, it is primarily provided to medical professionals or professional users. Professional users are persons who, based on their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or their prevention. The product should only be used by or under the guidance of professional users. Use by non-professional users or medical laypersons must not take place without sufficient instruction. The product and accessories may only be used in accordance with these operating instructions or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the integrity of the product before each use. In case of damage, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate proximity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointy objects.
- ! The product should not come into direct contact with painted or coated surfaces.
- ! Use the product only on dry, non-slip, straight surfaces, e.g. on seats of chairs, stools or benches. Do not use the product on soft surfaces, e.g. on seats of armchairs or sofas.
- ! Choose an exercise program that fits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! Use only accessories recommended by the manufacturer.
- ! Use by children, adolescents and during pregnancy as well as in case of existing health problems only under professional supervision and after consultation with a doctor.

 The product and its accessories are no toys and should be kept out of the reach of children.
Do not allow children to play with it.

Product information

Scope of delivery

One of the following products:

- consisting of: 1 x SISSEL® SITFIT® 33 cm, 1 x instructions for use, 1 x exercise poster
- consisting of: 1 x SISSEL® SITFIT® 36 cm, 1 x instructions for use, 1 x exercise poster
- consisting of: 1 x SISSEL® SITFIT® Plus, 1 x instructions for use, 1 x exercise poster, 1 x needle pump

The SISSEL® SITFIT® / SITFIT® Plus is available in red, black and blue. The SISSEL® SITFIT® has a round shape and is available in diameters of approx. 33 cm and approx. 36 cm. The SISSEL® SITFIT® Plus also has a round shape with thigh support extending on one side and approximately 37 cm in diameter.

Technical data

Diameter:	SISSEL® SITFIT®: approx. 33 cm / 36 cm; Height: approx. 6.5 cm SISSEL® SITFIT® Plus: approx. 37 cm; Height: approx. 6.5 cm
Weight:	SISSEL® SITFIT® 33 cm: approx. 0.9 kg SISSEL® SITFIT® 36 cm: approx. 1.1 kg SISSEL® SITFIT® Plus: approx. 1.14 kg
Material Casing:	recyclable PVC (polyvinyl chloride), phthalate-free
Material Filling:	air, variable pressure
Max. user weight:	138 kg

Intended use

The SISSEL® SITFIT® / SITFIT® Plus serves as a multifunctional air-filled seat cushion suitable for everyday use on a firm seat surface, which enables the correction of the sitting posture and the unconscious training of the back and abdominal muscles. The desired slight instability promotes active-dynamic sitting and a physiological sitting posture. The SISSEL® SITFIT® / SITFIT® Plus is versatile and can be used at home, at the workplace or in physiotherapy and rehabilitation for various medical indications. Dynamic sitting can be done independently under direct guidance or after instruction by qualified personnel. The SISSEL® SITFIT® / SITFIT® Plus is equally suitable as a training device for improving balance and coordination while sitting or standing. In addition, various exercises for strengthening the back, abdomen, pelvic floor, arm and leg muscles can be performed while sitting, standing and in the supine, lateral or prone position. The integrated valve in the cushion allows easy regulation of the air filling. Use a needle pump for this purpose. This allows you to individually regulate the degree of instability.

Note: You can add or remove air from the SISSEL® SITFIT® / SITFIT® Plus at any time using the needle pump supplied. To do this, carefully insert the needle tip of the pump into the valve. Be careful to avoid puncturing the opposite sleeve. This is made easier by supporting yourself on the SISSEL® SITFIT® / SITFIT® Plus with your knee or forearm. A bulge forms at the valve area so that puncturing is avoided. You can now supply air with the pump. To deflate, simply insert the needle into the valve and hold the needle slightly sideways until the desired amount of air has escaped.

Note: The SISSEL® SITFIT® is designed as a round seat cushion and is well suited for common sitting phases and gymnastic exercises. The SISSEL® SITFIT® Plus has a round side and a side with thigh support. If the round side is in front, the cushion is well suited for common sitting phases and gymnastic exercises. If the thigh support is in front, the cushion also allows longer sitting phases by supporting and relieving the thighs. Use the SISSEL® SITFIT® / SITFIT® Plus for only a few minutes at first to get used to it and then slowly increase the duration of sitting. Also change later between the usual sitting on a chair / office chair and the active-dynamic sitting on the SISSEL® SITFIT® / SITFIT® Plus. Depending on the outside or room temperature, the SISSEL® SITFIT® / SITFIT® Plus may initially feel a little cool. After a few minutes, however, it adjusts to the body temperature.

-  Therapeutic use should only be done under the supervision or instruction of qualified personnel.
-  Risk of injury! Only use the product if you can ensure correct performance of the exercise recommendations described. Supervision by qualified personnel is recommended.
-  Risk of injury! On very slippery or moist surfaces, the SISSEL® SITFIT® / SITFIT® Plus can slip away despite its high slip resistance. Therefore, always make sure that the surface is fully covered and dry.

Indication

Promotion and training of active-dynamic, anatomically correct, ergonomic sitting, treatment, relief and prevention of posture-related tension and back and lumbar spine complaints, relief of intervertebral discs, strengthening of deep-lying, stabilizing back muscles, relief of sitting, balance and coordination training. When used as a training aid: improve body awareness, balance, coordination, responsiveness and depth sensitivity. Strengthening of deep muscles, improvement of muscular coordination, joint stability and promotion of psychomotor skills.

Contra Indication

Lack of compliance (due to fear or pain), acute disc or lumbar spine symptoms, acute injuries of the lumbar spine (e.g. herniated disc, inflammations), open wounds, unstable fractures of the spine, acute radiating pain, paraesthesia, movement disorders in the lumbar spine and legs, unexplained disease findings, pronounced balance disorders, impairment due to fatigue, medication, alcohol.

Regulatory requirements

The current product of SISSEL® GmbH complies with the following regulatory requirements:

- Medical Devices Regulation (EU) 2017/745

 All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessory

The product may only be used with the accessories intended for this purpose.

Optional accessories: needle pump and slipcovers available.

Usage

1. Check the packaging and the product for any damage upon delivery.
 Never use a damaged product.
2. Check the product for integrity and contamination before each use.
3. Use the product as intended.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & Hygiene).

Care & Hygiene

This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free operation must be ensured. The product can be wiped with commercially available cleaning agents and disinfectants. Use a soft cloth and no abrasive agents. Do not use agents with phenol content or those containing the substance diethylene glycol monoethyl ether.

-  Only use suitable disinfectants that have been tested for effectiveness to carry out disinfection measures.
-  Disinfectants that can damage the materials of the product must be avoided.
-  The application instructions of the disinfectant manufacturer must be followed.

Storage: Store the SISSEL® SITFIT® / SITFIT® Plus in a clean place without direct sunlight. Sunlight can cause discoloration of the surface material. Discoloration does not affect the function.

The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee claims.

Disposal & Environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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SISSEL GmbH