The natural way of Sweden

SISSEL® SECUREMAX® BALL

- ▮ Gebrauchsanweisung
- ▮ Instructions for use
- ▮ Mode d'emploi
- ▮ Gebruiksaanwijzing
- ▮ Instrucciones de uso
- ▮ Istruzioni per l'uso
- ▮ Bruksanvisning
- ▮ 사용설명서

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General information

Read the following operating instructions carefully and follow the safety instructions and warnings. The SISSEL® SECUREMAX® Ball is an air-filled seat for promoting a correct sitting posture and for versatile use in balance and coordination training, fitness training and for performing more varied gymnastics. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the system or how to avoid them. The therapeutic use of the SECUREMAX® ball, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these operating instructions or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the condition of the product before each use. If damaged, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointy objects.
- ! Train only on dry, non-slip and straight surfaces and not near stairways, balustrades, edges, etc.
- ! Choose an exercise program that fits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk. Do NOT stand on the ball.
- ! Use only accessories recommended by the manufacturer.
- ! Use by children, adolescents and during pregnancy as well as in case of existing health conditions or cognitive impairment only under supervision and after medical consultation.
- ! The SECUREMAX® ball must only be filled with air. A commercially available hand or ball pump is recommended for filling the SECUREMAX® ball.
- ! Do not over-inflate the ball. An over-inflated ball can burst during use and cause injury.
- ! Only inflate the ball as described in the instructions for use. Improper inflation can cause damage to the ball and cause injuries.
- ! The product and its accessories are no toys and should be kept out of the reach of children. Do not allow children to play with it. Risk of strangulation and suffocation.

Product information

consisting of: 1 x SISSEL® SECUREMAX® Ball, 2 x sealing plugs, 1 x user manual, 1 x training poster, 1 x measuring tape, 1 x plug lifter

The SISSEL® SECUREMAX® Ball is available in four different variants (45 cm, 55 cm, 65 cm, 75 cm) and four different colors (blue-purple, red, silver and green).

Technical specifications

| | |
|-------------------|---|
| Dimensions (Ø): | 45 cm, 55 cm, 65 cm, 75 cm |
| Colors: | blue-purple, red, silver, green |
| Material Cover: | recyclable PVC (polyvinyl chloride), phthalate-free |
| Filling: | air, variable pressure |
| max. user weight: | 150 kg |
| max. load: | 500 kg |

Note:

- The nature of the cover material prevents the ball from bursting in the event of sudden punctual damage. The air escapes slowly in these cases when the max. user weight is observed, sufficient reaction time remains and the risk of accidents is minimized.
- Gymnastic balls generally lose air slowly over time. The diameter of the SISSEL® SECUREMAX® Ball decreases by approx. 2 to 3 cm per month. This is normal and does not represent a defect.

Intended use

- The SISSEL® SECUREMAX® Ball serves as an everyday multifunctional seat for the correction of the sitting posture and the unconscious training of the back and abdominal muscles. Furthermore, the SISSEL® SECUREMAX® Ball is a universal sport and training device for the extension of general fitness training, for targeted training in certain sports and serves especially physiotherapeutic purposes in the prevention, treatment and aftercare of various clinical indications. Due to the desired variable instability of the SISSEL® SECUREMAX® Ball, physiotherapeutic programs for training beginners or inexperienced users up to professional training concepts can be suitably supported.
- The SISSEL® SECUREMAX® Ball is multifunctional and can be used at home, at the workplace or in physiotherapy and rehabilitation for various medical indications. The active-dynamic sitting can be done independently, under direct guidance or after instruction by qualified personnel. (For example exercises see poster insert)
- With a standard ball pump, the pressure of the SISSEL® SECUREMAX® Ball and thus the difficulty of the exercises can be easily increased or decreased.

Preparation:

- Before the first inflation, the ball should lie spread out for 24 hours and adjust to the room temperature (ideal: 23 °C). This simplifies the inflation and prevents overstretching.
- Inflate the ball in a number of steps to 80 % of the intended diameter (check with the enclosed measuring tape). In this way, you avoid overstretching the material. Use a hand pump (optional accessory) or a commercially available ball pump for inflation.
- Let the ball rest for 2 hours. The special material needs this time to adjust to the stretch and internal pressure. Then inflate the ball to the desired size and strength (check with enclosed measuring tape). Do not exceed the specified ball diameter.

- Due to the special material, the ball may initially feel a little hard and the material may have slight wrinkles. This is normal and settles after a short time.
- After final inflation, allow the SISSEL® SECUREMAX® Ball to rest at room temperature for 24 hours before first use.

Important: The sealing plug of the air inlet must be inserted into the ball as far as it will go. To pull out the sealing plug, please only use the enclosed plug lifter.

- During use, the SISSEL® SECUREMAX® ball can change its shape slightly. At rest, the ball soon returns to its original shape.

 Risk of injury! Only use the product if you can ensure correct performance of the exercise recommendations described. Supervision by qualified personnel is recommended.

Indication

Support of active-dynamic sitting, support of anatomically correct upright, ergonomic sitting. Treatment, alleviation and prevention of postural tension and back and lumbar discomfort, relief of intervertebral discs, strengthening of deep stabilizing back muscles, balance and coordination training. Improvement of body awareness, strengthening of deep muscles, improvement of muscular coordination, reduction of postural deficits. Pre- and postoperative care for musculoskeletal surgeries, support for rehabilitative measures. Extension and optimization of the fitness program.

Contra Indication

Pronounced sensory perception and balance disorders, lack of compliance (e.g. due to fear or pain), unexplained disease symptoms, acute lumbar spine syndrome, acute injuries or inflammations of the spine (herniated discs), unstable fractures of the spine or pelvic region, acute radiating pain, sensory disturbances, movement disorders, impairment due to fatigue, medication, alcohol.

Regulatory requirements

The current product of SISSEL® GmbH complies with the following regulatory requirements:

- Medical Devices Regulation (EU) 2017/745

All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessory

The product may only be used with the accessories intended for this purpose.

Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

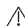
2. Check the product for integrity and contamination before each use.

3. Use the product as intended.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & Hygiene).

Care & Hygiene

- This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free operation must be ensured.
- The product can be wiped with a damp cloth and then rubbed dry.
- The product can be wiped off with commercially available surface disinfectants.
- Use a soft cloth and no abrasive agents. Do not use agents with phenol content or those containing the substance diethylene glycol monoethyl ether.

 Only use suitable disinfectants that have been tested for effectiveness to carry out disinfection measures.

 Disinfectants that can damage the materials of the product must be avoided.

 The application instructions of the disinfectant manufacturer must be followed.

The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee claims.

Disposal & Environment

Product: The product and the accessories supplied with it can be disposed of with household waste.

Packaging: No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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