
SISSEL[®] SILENCIUM[®]

- ▮ Gebrauchsanweisung
- ▮ Mode d'emploi
- ▮ Gebruiksaanwijzing
- ▮ Instrucciones de uso
- ▮ Instructions for use
- ▮ Istruzioni per l'uso
- ▮ Bruksanvisning

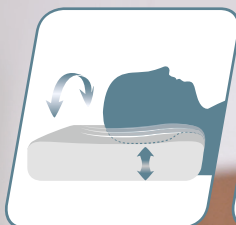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EVERYONE DESERVES A SISSEL® SILENCIUM®

Congratulations on your SISSEL® SILENCIUM® purchase. You now belong to a growing number of users worldwide benefiting from a significant reduction of snoring and enjoying a peaceful and restful sleep from now on. SISSEL® SILENCIUM® was designed to enable snorers and their partners to have a quiet and deep night's sleep. Please read through these instructions carefully before initial use to understand the functions, effects and benefits of the SISSEL® SILENCIUM®.

EFFECTS OF THE SISSEL® SILENCIUM®

SISSEL® SILENCIUM® continuously records the sleeping sounds and your head position on the pillow. It turns your head independently, quietly and gently to the side when a snoring is detected. Turning of the head increases the gap between the tongue and throat and thus expands the airways and reduces snoring. If no positive effect is achieved with the first movement, the SISSEL® SILENCIUM® further turns the head step by step: to the left or right, up and down until the snoring stops or the ideal position is found in which the snoring is reduced the most.

INDICATIONS

- normal snoring (medical: compensated snoring) without clinical significance

CONTRA-INDICATIONS

- presence of postural vertigo
- persons with suspected sleep apnea syndrome should consult a doctor prior to use
- not recommended for use by children
- persons with pacemakers should consult a doctor before use

GENERAL COMMENT

Snoring is generally subject to the individual's health and body condition. Alcohol consumption, eating habits, colds, stress, medication and the physical/mental condition may additionally influence snoring habits, intensity and duration as well as your individual body mass index (BMI). Please take this into consideration.

NOTE


If you notice signs of fatigue and lack of sleep after using the pillow, this could be a sign of further problems, such as sleep apnea. In this case, please consult your doctor.

After unpacking the SISSEL® SILENCIUM®, a temporary characteristic odor might occur. This smell is completely harmless and will disappear after a short time. In order to avoid any unpleasant odor, air the product before use and keep the room ventilated for a while.

ADVANTAGES

- reduces snoring significantly (result of a clinical study)
- supports a healthy sleep – of those affected as well as partners
- easy to use – like an ordinary pillow
- thoroughly tested

COMPOSITION/COMPONENTS OF THE SISSEL® SILENCIUM®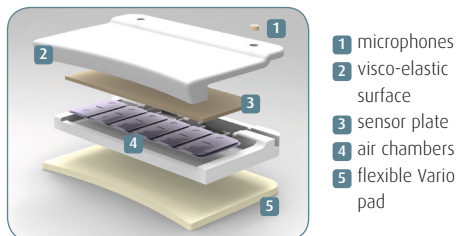

SISSEL® SILENCIUM® provides several functions. It is a height-adjustable pillow made of high-quality polyurethane foam with a **visco-elastic surface (2)** for utmost comfort and a **flexible Vario pad (5)** to adjust the pillow height.

The **embedded air chambers (4)** inside the pillow gently guide your head in the desired position as soon as a snoring sound occurs. The **sensor plate (3)** registers the head position. For the protection of the **microphones (1)**, the pillow has an air-permeable, moisture-repellent inner protective cover. On top comes the washable pillowcase.

GETTING USED TO THE SISSEL® SILENCIUM®/ADJUSTMENT PERIOD

Users might require some time (approx. 28 days) to adjust to the special product features of the SISSEL® SILENCIUM® (e.g. changing of head position). For easier adjustment, the Vario pad can be removed. Depending on the positioning of the control unit (distance to the head), some sound may initially be registered upon activation of the air chambers. We therefore recommend to place the control unit in the supplied case and as far away from the head as possible under the bed (or in the bedside drawer) – if necessary, additionally wrapped in a towel. Once accustomed to the pillow, users will usually not perceive these sounds as disturbing anymore.

ELECTROMAGNETIC COMPATIBILITY (EMC)

SISSEL® SILENCIUM® operates with safe low-voltage (power supply: 6 V) and, due to its classification as a medical device class I (rule 12), is also suitable for use in hospitals, rehabilitation centers etc. This voltage is comparable to a clock radio and lower than the voltage of an electric blanket. Using a DC adapter, the SISSEL® SILENCIUM® can also be used on vacation (e.g. mobile home).


BEFORE USING THE SISSEL® SILENCIUM®

Make sure that SISSEL® SILENCIUM® is used in a clean and dust-free environment with low humidity. Dust and moisture can limit the lifetime and functionality of the technology.

Do not use the pillow near electric blankets, hot water bottles or other hot objects, as their high temperatures can damage the pillow or its properties.


USE OF THE SISSEL® SILENCIUM® – INITIAL OPERATION

Place the SISSEL® SILENCIUM® on your bed like a normal pillow (see figure).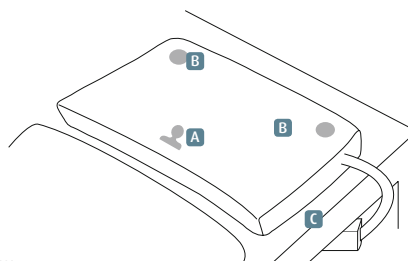

Pillow:

- A** symbol for correct pillow position
- B** microphones
- C** positioning of control unit in case

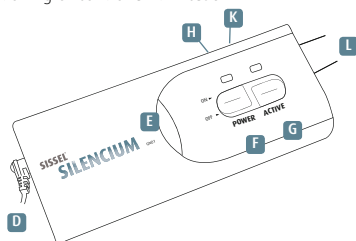Control unit:

- D** power plug (6V DC)
- E** system display: **GREEN LIGHT UP**, if the pillow is exposed to pressure (head placement was registered)
GREEN FLASHING, if snoring sounds are recorded
- F** power ON/on switch
- G** active ON/air chamber activity ON (anti-snoring function activated)

Active ON – the air chamber activity is activated. Snoring sounds are recorded and the air chambers embedded in the cushion softly guide the head of the user in a different position until the snoring stops.

- H** USB connection (optional)
- K** line-IN for external microphone (optional)
- L** cable to pillow

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Do not expose the pillow to excessive pressure by forcibly squeezing, pushing or bending it. Make sure that the two microphones (B) at the front of the pillow are not covered. If desired, you can additionally use any conventional pillow case (does not affect the microphone output) for the SISSEL® SILENCIUM®. Plug the power plug (D) in the power connector and the control unit (C). Turn both switches, power (F) and active (G) ON. Both lights on the control panel light up green. Place the control unit with case at the side of or under your bed. Make sure that the cable between the control unit and the pillow is not tightened or pinched. SISSEL® SILENCIUM® was developed for use by a single person.

TRANSPORT AND STORAGE

Store the SISSEL® SILENCIUM® in its original shape in a dry environment (maximum 65% relative humidity) and at a suitable room temperature (+5 to +45°C). Store the SISSEL® SILENCIUM® carefully (do not fold or bend, do not expose to pressure) to prevent shape or technical damage of the pillow. Use the original packaging for transportation.

CLEANING AND MAINTENANCE

The pillow cannot be washed or dry cleaned. To clean the pillow, remove the outer cover, air the pillow or gently wipe the inside, moisture-resistant protective cover with a slightly damp cloth to remove dust or stains from the surface. The inner cover protects the sensitive technical parts of your pillow from moisture buildup.

The outer pillow cover can be removed and cleaned according to the instructions on the washing label (inside the cover). For washing, the cover needs to be turned inside out and the zipper closed.

WARRANTY OF THE SISSEL® SILENCIUM®

SISSEL® SILENCIUM® is issued with a 2-year product warranty. The warranty will only be granted if the pillow was purchased from an authorized dealer or shop.

GENERAL CONDITIONS OF WARRANTY

- The warranty is only valid if the SISSEL® SILENCIUM® was used according to the guidelines stated in the instructions for use.
- The warranty covers electrical and mechanical defects and permanent deformation of the foam of more than 2 cm of the original shape.

- In case of problems or complaints, please contact the authorized dealer from whom you purchased the SISSEL® SILENCIUM®. For warranty questions, please submit the original invoice or the original proof of purchase.
- If the SISSEL® SILENCIUM® was replaced or repaired during the warranty period, the original warranty period is not extended.

THE WARRANTY DOES NOT APPLY

- If the instructions for use, cleaning and maintenance of the SISSEL® SILENCIUM® stated in this manual were not followed or if it was damaged in any other way by poor maintenance or careless use.
- If the pillow or other components were forcibly compressed, bent or squeezed or exposed to any other excessive force resulting in a material failure, loss of functionality or permanent deformation of the pillow.
- If the pillow or its components get wet or are soaked.
- If the pillow or cover is soiled or unhygienic.
- If the pillow or other components have been modified or repaired without written permission.
- If the pillow and its components, especially the air pump, pipes and air valves have been damaged by high humidity or dusty environment.
- If the cover has not been washed according to the instructions on the label, which is located on the inside.

THE RETURN MAY BE REJECTED:

- If the contents of the product packaging are incomplete.
- If the pillow or its components were damaged or improperly – i.e. not in accordance with the instructions for use – used.

ADDITIONAL PRODUCT INFORMATION

Application environment:

- temperature: 5°C – 45°C
- relative humidity: ≤ 65 %

Technical specifications:

- power supply pillow: 6 volts DC 
- voltage power supply: 220V
- line voltage: refer to the domestic standard
- class 2
- pillow size: ~ 60 x 40 x 7/9 cm
- pillow material: PU foam with comfortable visco-elastic layer

Scope of delivery:

- SISSEL® SILENCIUM® anti-snoring solution with separate Vario insert
- power supply
- instructions for use
- outer cover 40 % viscose/60 % polyester, washable at 60°C
- case

NOTE

The SISSEL® SILENCIUM® anti-snoring solution is continuously refined and improved. There might be slight variations which do not influence the function.

SISSEL® SILENCIUM® complies with the regulations of the directive 93/42/EEC for medical devices and corresponds to class I (rule 12).

FAQ – Please also take a look at our FAQ list (frequently asked questions) with interesting user information at www.sisselsilencium.com

Manufacturer: SISSEL® GmbH, Bruchstrasse 48
D-67098 Bad Dürkheim/Germany
www.sissel.com
Hotline: +49 (0)6322/98943-0

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Producent: SISSEL® GmbH