The natural way of Sweden

SISSEL® STEP-FIT®

- Gebrauchsanweisung
- Instructions for use
- Mode d'emploi
- Gebruiksaanwijzing
- Instrucciones de uso
- Istruzioni per l'uso
- Bruksanvisning

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General information

Read the following user manual carefully and observe the safety instructions and warnings.

The SISSEL® STEP-FIT® is a training device for the promotion of sensory motor skills, coordination and balance, as well as relaxation through a pleasant foot reflex zone massage.

The product may only be used in accordance with these instructions for use. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these user instructions.

The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on.

Safety instructions

- ! Check the integrity of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- ! Do not use the product in the immediate vicinity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the exercise area free from sharp objects or edges.
- ! Exercise only on dry, non-slip, straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a rubberised bottom (anti-slip socks). Do not train with shoes and make sure you wear suitable sports clothing.
- ! Choose a training programme that suits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.
- ! Only use accessories intended by the manufacturer.
- ! Use in children and adolescents, during pregnancy and in case of existing health problems such as dizziness, cardiovascular problems, joint weaknesses, etc. only under professional supervision and after consulting a doctor.
- ! The product and its accessories are not toys and must be kept out of the reach of children. Do not allow children to play with it.

Product information

Scope of delivery consisting of:

1 x SISSEL® STEP-FIT®, 1 x Instructions of use

Technical data

Measurements (L x W x H):	49 cm x 49 cm x 2 cm
Weight:	ca. 2000 g
Material:	Thermoplastic elastomer
Max. User weight:	120 kg
Colour:	grey

Intended use

- Walking barefoot is the simplest and most natural form of foot reflex zone stimulation. Walking on pebbles reinforces this; with each step, nerve points are stimulated, the soles of the feet are massaged and blood circulation is improved.
- Stimulating the foot reflex zones by walking on the SISSEL® STEP-FIT® can have a positive effect on e.g. digestion, sleep rhythm etc. and strengthen the back muscles. SISSEL® STEP-FIT® offers barefoot walking in an optimised form – hygienic, skin-friendly and safe!

⚠ Warning: Risk of injury! Only use the product if you can ensure that the exercise recommendations described are carried out correctly. In case of balance disorders, consult your doctor or physiotherapist before exercising with the SISSEL® STEP-FIT®.

⚠ Warning: Risk of injury! Do not use the product on open wounds and/or injuries on the feet and soles.

Usage

Check the packaging and the product for any damage on delivery.

⚠ Warning: Never use a damaged product.

Application examples:

Integration in the course suitable for balance training, muscle building, whole body coordination and others.

Goal:

The training/improvement of the above-mentioned areas by bringing about different surfaces and levels of difficulty.

Areas of application:

- Sensorimotor and proprioceptive training
- Strengthening of the musculature (throughout the body)
- improves whole-body coordination
- Sensitivity and perception training (can also be combined with other surfaces on the course).
- Maintaining physical fitness and performance
- Balance training
- Stimulation of the metabolism
- can improve blood circulation

Warning: After use, cleaning and possibly disinfection measures may be necessary (see Care & Hygiene).

Care & Hygiene

The product can be wiped off with commercially available cleaning agents and disinfectants. Use a soft cloth and no abrasive agents. Do not use any agents containing phenols or diethylene glycol monoethyl ether.

⚠ Warning: Disinfectants that can damage the materials of the product must be avoided.

⚠ Warning: The application instructions of the disinfectant manufacturer must be observed.

Disposal & Environment

- Product: The product can be disposed of with household waste.
- Packaging: No special types of packaging are used. The respective valid disposal guidelines must be observed.

Warranty

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed.

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SISSEL® GmbH • Bruchstrasse 48