*The natural way*

# SISSEL® THERM

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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## General information

Read the following operating instructions carefully and follow the safety instructions and warnings. The SISSEL® THERM is a gel-filled, self-activating heat pad for targeted, localized heat therapy. As a medical device, it is also sold to medical professionals and professional users for therapeutic purposes. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge of the diseases to be treated with the system or how to avoid them. The therapeutic use of heating pads, e.g. in physiotherapy and rehabilitation treatment, should be carried out by or under the guidance of professional users if possible. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these operating instructions or as instructed by appropriately trained specialists. SISSEL® GmbH accepts no liability for the use of the system outside the applications described in these instructions for use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. The regulatory requirements applicable to this product in the respective country of use must be taken into account.

## Safety instructions

- ! Check the undamaged condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed, escaping hot gel can cause skin burns.
- ! Heat the pad for reprocessing only in a boiling water bath. Do not heat the pad in the microwave, oven or grill.
- ! Carefully remove the pad from the water bath after it has been completely heated. Danger of burns!
- ! Knead the pad well after activating it. This distributes the heat evenly in the gel and prevents heat pockets from forming (risk of burns!).
- ! Always place the pad carefully directly on the skin! If necessary, use a thin cotton cloth or a piece of clothing to avoid irritation or damage to the skin.
- ! Check the temperature by carefully holding the pad against the inside of the wrist.
- ! Remove the pad immediately if you feel uncomfortable with the heat.
- ! Children or persons with cognitive impairments should use the pad only under supervision.
- ! Leaking, damaged or deflated pads must not be used. Danger of burns!
- ! The contents of the compress are not suitable for consumption.
- ! Only use accessories intended by the manufacturer.

 The product and its accessories are not toys and must be kept out of the reach of children.  
Do not allow children to play with it.

## Product information

### Scope of delivery

consisting of: 1 x SISSEL® THERM, 1 x instructions for use

### Technical data

Dimensions (L x W): approx. 42 cm x 18 cm

Material cover: PVC film, phthalate free

Material filling: sodium acetate, water, dye

**Note:** The special 3-part fit of the SISSEL® THERM is ideal for deep, effective heat application in the shoulder-neck area or for the lumbar spine.

### Intended use

The SISSEL® THERM pad provides drug-free, on-demand, individualized heat therapy. Heat applications use the reactions of the skin, subcutaneous tissue, muscles and deep tissue layers to the temperature stimulus. In heat therapy, these are circulation-enhancing and metabolism-stimulating effects. Thus, the pad is an ideal way to use heat in a targeted therapeutic manner on various occasions.

**Note:** The gel contained in the pad is non-toxic. In case of contact of the gel with the skin, eyes or mucous membranes of the mouth and nose rinse with plenty of clean water. Skin and mucous membrane irritations are not known.

### Indication

Tension, muscle soreness, myogelosis/hard tension, back pain, tendopathies, muscle and tendon shortening, lumbago, menstrual cramps, myalgias, gastrointestinal tract irritations, hypertension (high blood pressure), general nervousness and restlessness, colds. If in doubt, seek medical advice.

### Contra Indication

Acute inflammation, acute trauma (accidents, open wounds), acute joint inflammation, edema, febrile infections, dermatitis, deep vein thrombosis, peripheral arterial occlusive disease, hypotension, heart disease, impaired skin sensation (e.g. diabetics, heat/cold insensitive persons, children under 3 years). If in doubt, seek medical advice.

### Regulatory requirements

The current product of SISSEL® GmbH complies with the following regulatory requirements:

- Medical Devices Regulation (EU) 2017/745

 All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

### Accessories

The product must only be used with the accessories provided.

### Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

2. Check the product for integrity and contamination before each use.

3. Carefully bend the activator leaflet to activate the product.

The SISSEL® THERM will now automatically heat up to approx. 50 °C to 55 °C in less than 30 seconds.

4. Now knead the pad well so that the heat is evenly distributed in the gel.

 Risk of injury! The heat may be unevenly distributed at first. It is important to knead the pad well after each activation to distribute the temperature evenly and avoid heat pockets. Danger of burns!

5. Before each application, check the temperature of the activated pad on the inner surface of the wrist. If you feel the heat is too intense, let the pad cool down a little or use a thin cotton cloth or wear a thin piece of clothing as an intermediate layer. Danger of burns!
6. Carefully place the pad directly on the skin, if necessary use a thin cotton cloth or wear a thin piece of clothing. After activation, the SISSEL® THERM emits heat for about 60 minutes.

#### Reprocessing:

1. Place the completely cooled pad in a pot of boiling water for approx. 10 to 20 minutes until the crystals of the filling have completely liquefied and the filling is transparent.  
Note: Make sure that all the crystals are completely melted, otherwise the entire contents of the pad will crystallize again after cooling.
2. Carefully remove the pad from the boiling water (risk of burns!) and allow it to cool completely before use.  
Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & Hygiene).

#### **Care & Hygiene**

This product is suitable for repeated use. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and error-free operation must be ensured. In case of soiling, the compress can be wiped off with a soft cloth and lukewarm water. Then leave to dry in the air. Disinfectants, scouring agents, glass cleaners or all-purpose cleaners damage the surface materials of the product and must therefore not be used.

Note: The use of unsuitable products or products that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee.

Storage: Protect from direct sunlight. Store SISSEL® Therm in a dry, dark and clean place.

#### **Disposal & Environment**

The product and the accessories supplied can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be observed.

#### **Warranty**

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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SISSEL GmbH