*The natural way of Sweden*

# SISSEL® PORTABLE MASSAGE TABLE BASIC

- ▮ Gebrauchsanweisung
- ▮ Instructions for use
- ▮ Mode d'emploi
- ▮ Gebruiksaanwijzing
- ▮ Instrucciones de uso
- ▮ Istruzioni per l'uso
- ▮ Bruksanvisning
- ▮ 작동 매뉴얼
- ▮ Εγχειρίδιο Χρήσης

## SISSEL® KOFFERMASSEBANK BASIC

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## SISSEL® PORTABLE MASSAGE TABLE BASIC

The SISSEL® Basic portable massage table is a practical, functional and stable massage table which can be easily transported.

Please read the instructions for use carefully and retain them for the entire lifetime of the product and observe the safety and warning information.

The portable SISSEL® massage table can be used for massages in the comfort of your home, at exhibitions and company events and wherever a portable massage table is useful.

### Note:

For medical use we recommend the purchase of purpose-built treatment tables.

### Areas of Application:

- Massage
- Wellness or beauty treatments

### Overview of the Main Features:

- Foldable with flexible solid wood struts and coated cables
- Wooden parts made from solid- and laminated beech.
- An opening in the front strut simplifies client access, the rear strut adds extra stability.
- Working weight approx. 140 kg, static weight approx. 250 kg
- Load-bearing capacity tested according to EN 1725:1998 and EN 1730:2012.
- Washable skin-friendly PU surface, PVC free
- Upholstery: multi-layered PU foam, a viscose foam layer for comfort, rounded and reinforced corners, height approx. 5cm
- Carry case
- Measurements: approx. 186 x 62 cm
- Height: adjustable from approx. 58 cm to 87 cm

### Transport and Storage:

Store the massage table securely in the carry case in a dry place. Ensure that the massage table does not get in contact with sharp objects or harmful materials.

### Hygiene and Cleaning:

Before use, cover the surface with a towel, a loose soft cover or a fitted cover. Wipe the table with neutral soap or a mild disinfectant that contains no more than 30% alcohol. Apply with a damp cloth and then dry with a clean one.

### Technical Data:

- Weight: ~14 kg
- Material: beech
- PU cover

### Warning notice:

- Check the SISSEL® Basic massage table before each use.
  - All screws and knobs must be securely fastened.
  - Ensure that there are no cracks in the wooden parts.
  - The cables must all be intact.
- The maximum load should not exceed 250 kg.
- The feet of the massage table must be adjusted to the same height.
- Ensure the table has a secure position.
- Do not insert fingers into the moving parts or joints.
- Young people especially babies and toddlers should not be left unsupervised on the table. This product is not suitable for non-responsive people (risk of falling).

### Assembly:

- 1 Unlock the case.
- 2 Open up the table. Remove accessories and lay them out. Straighten each joint out as far as possible. Press the strut slightly downwards.
- 3 Lift the table up and set it up.
- 4 Pull the table at the head end towards you.
- 5 Press the table down in the middle.

### Adjusting the Height: 6

- Loosen screws, adjust legs to desired height and re-tighten securely.
- The feet of the massage table must be adjusted to the same height.

### Dismantling:

- Lay table on its side, completely fold the legs flat under the table and fix all accessories and cables under the table.
- Close all fasteners.
- When closing the table please ensure that cables are correctly positioned and accessories removed.

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SISSEL® GmbH