ooc
SPINEFITTER
BACK IN BALANCE

SPINEFITTER®
by SISSEL®
Junior

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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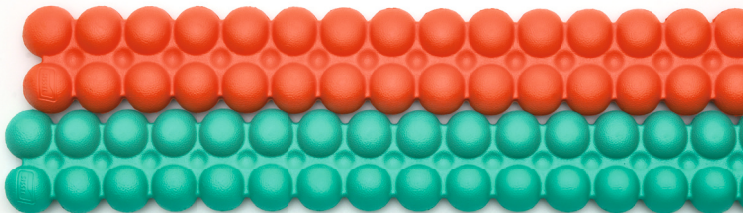
SPINEFITTER® by SISSEL® Junior

General information

Read the following instruction for use carefully and observe the safety instructions and warnings. The SPINEFITTER Junior is a product for children from the age of four and teenagers to support them in training, physiotherapy, prevention and rehabilitation. As a medical device, it is also supplied for therapeutic purposes to medical circles or professional users. Professional users are persons who, due to their medical or comparable training, have sufficient knowledge regarding the diseases to be treated with the product or how to avoid them. The therapeutic use of the SPINEFITTER Junior, e.g. in physiotherapy or rehabilitation treatment, should preferably be carried out by or under the guidance of professional users. Autonomous training by non-professional users and medical laypersons is possible under consideration of the instructions for use. However, initial instruction by appropriately trained professionals is recommended. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL GmbH accepts no liability for the use of the product outside the applications described in these instructions for use. The instructions for use must be kept carefully and within easy reach, and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the condition of the product before each use. If damaged, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate area of heat sources (e.g. radiator, hotplate, open flame, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointed objects.
- ! Train only on dry, non-slip and straight surfaces and not near stairways, balustrades, edges, etc.
- ! Train exclusively barefoot or wear socks with a non-slip base. Do not train with shoes and make sure you wear suitable sports clothing.

 Risk of injury! Choose a training programme that suits your physical strength and fitness level and avoid overly difficult exercises or experiments with increased risk.

- ! Only use and store the SPINEFITTER flat and do not place any objects on it.
- ! Do not use the SPINEFITTER with more than one person at the same time.
- ! Use only accessories recommended by the manufacturer.
- ! Use in the presence of health issues such as dizziness, heart/circulatory problems, joint weaknesses, etc., only under professional supervision and after consulting a doctor.

Product information

Scope of delivery

consisting of: 1 x SPINEFITTER by SISSEL Junior, 1 x instruction for use, 1 x training poster

Technical data

Dimensions (L x W x H): approx. 72 x 9,5 x 4,5 cm

Material: Closed-cell, cross-linked foam

X:: EXTRALIGHT®

Minimum age of user: 4 years

Recommended maximum height: 160 cm

Maximum user weight: 60 kg

Intended use

The SPINEFITTER Junior is an innovative therapeutic aid and training device for children from the age of four and teenager, which can be used in the treatment of muscular, sensory and coordination deficits, as well as for balance exercises, psychomotor training and to develop body awareness. The SPINEFITTER Junior's highly motivating character and wide range of uses make it easy to carry out simple training and targeted therapeutic measures. This makes it the perfect partner for supporting and promoting children and young people in physiotherapy and sports.

 Risk of injury! Only use the SPINEFITTER Junior if you can ensure that the described exercises are performed correctly. Supervision by a specialist is recommended. Children and teenager must be supervised during use.

Indication

- Lack of core strength
- Postural deficiencies and errors
- Poor body awareness and body perception
- Sensory integration disorders (problems with fine motor skills, difficulties with gross motor skills, clumsiness, movement planning disorders)
- Scoliosis
- Motor development disorders
- Coordination and balance disorders
- Ankle instability
- Insufficient straightening of the arch of the foot (flat foot/splayfoot)
- To improve posture, body awareness, body perception, movement sequences, balance, general coordination, psychomotor skills, coordination and sensory integration

Contra indication

- Acute injuries (open wounds, soft tissue injuries, unstable fractures in the area of the spine, pelvis, extremities, etc.)
- Acute pain and movement disorders
- Acute inflammatory diseases, fever
- Neurological diseases
- Non-physiological, age-inappropriate hypermobility
- Pronounced scoliosis requiring the use of a corset
- Rheumatic diseases

Regulatory requirements

The SPINEFITTER Junior of SISSEL GmbH complies with the following regulatory requirements:

- Medical Devices Regulation (EU) 2017/745
- Test according to EN 71-3:2019+A1:2021 EN 71-3:2019+A1:2021 (Safety of toys – Part 3: Migration of certain elements)

 All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessory

The SPINEFITTER Junior may only be used with the accessories intended for this purpose.

Usage

1. Check the packaging and the product for any damage upon delivery.

 Never use a damaged product.

2. Check the product for damage and contamination before each use.
3. Warm up before exercising.
4. Use the product as intended.
5. Exercise according to the exercise instructions or as instructed by professionals.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & hygiene).

Care & hygiene

This product is suitable for reuse by additional persons. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed to ensure a safe operation. Clean the SPINEFITTER Junior with a soft cloth and mild soap solution. Then wipe dry with a soft cloth. Visible soiling must be removed beforehand. The surface of the SPINEFITTER Junior can be disinfected by wiping with non-alcoholic products. Disinfectant products containing corrosive substances (e.g. acids, alkalis) should be avoided as they can damage the surfaces.

⚠ Only use suitable disinfectants that have been tested for effectiveness to carry out disinfection measures.

⚠ Disinfectants that can damage the materials of the product must be avoided.

⚠ The application instructions of the disinfectant manufacturer must be followed.

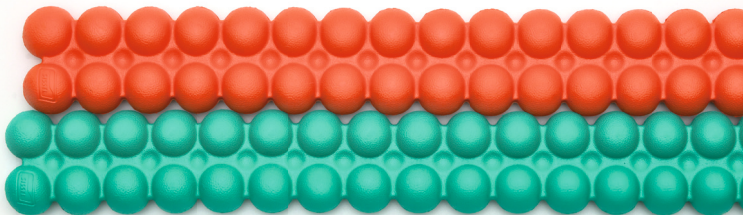
The use of unsuitable preparations or preparations that have not been tested for effectiveness or their incorrect use will void the warranty and guarantee claims.

Disposal & environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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