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# **SISSEL® Comfort – Product information**

## General information

- Read the following instructions for use carefully and observe the safety instructions and warnings.
- The SISSEL® Comfort support cushion is a multi-use support aid for assisting positioning for numerous applications.
- The product and accessories are to be used only in accordance with these instructions for use.
- SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions for use.
- The instructions for use must be kept carefully and within easy reach and must be handed to the next user if the product is passed on.

## Safety instructions

- ! Check the condition of the product before each use. If damaged, the function and safety of the product can no longer be guaranteed.
- ! Do not use the product in the immediate vicinity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free from sharp or pointed objects.
- ! Make sure that children do not have access to the inner components of the product.
- ! The use of the product as a baby nest should only be done under adult supervision.
- ! Always use the product with the supplied cover and close the zip carefully
- ! Only use accessories specified by the manufacturer.

## Product information

- Scope of delivery contents:
  - 1 x SISSEL® Comfort, 1 x instruction manual
- Available with two different filling materials:
  - polystyrene and polypropylene

## Technical specifications

- Dimensions (LxWxH): 195 x 35 cm
- Filling: polystyrene or polypropylene
- Cover: 100% cotton

## Note

The product is available with two different filling materials: polystyrene and polypropylene. The filling with polystyrene makes the product mouldable, the filling with polypropylene gives the product more dimensionally stability

## Intended use

The SISSEL® Comfort is a positioning pillow that can be used as a positioning aid for numerous applications. The curved shape ideally supports the neck and shoulder area, stabilises the position for people sleeping on their sides and provides optimal relief support for pregnant women and nursing mothers.

## Warning

-  Never use a damaged product.

## Care & Hygiene

- For cleaning, it is usually sufficient to wash the cushion in the bathtub. For machine cleaning, use a laundry bag (no net). Alternatively, the cushion can be washed in a separately available cover (please wash this separately the first few times and follow its cleaning instructions). Make sure that the cushion has enough room to move in the washing machine and dryer.
- Please be sure to read the attached label
- Cushions with polypropylene filling are machine washable up to 95°C
- Cushions with polystyrene filling are machine washable up to 60°C
- After washing, the cushions can be tumble dried at a low temperature

## Disposal & Environment

The product and the accessories supplied can be disposed of with household waste.

## Packaging

No special types of packaging are used. The respective valid disposal guidelines must be observed.

## Warranty

- The manufacturer accepts no liability for damage owing to any filling leaking out as a result of washing without cover or overfilling a washing machine.
- The warranty is governed by the statutory provisions.

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