SISSEL® Soft Curve Kissen

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SISSEL® Soft Curve Pillow

The gentle support pillow.

Congratulations on the purchase of this quality product. Compliance with our care and use instructions will guarantee continued daily enjoyment.

Positioning

Place the flat end of your SISSEL® Soft Curve under your head. The higher end should be at your shoulder. This ensures gentle support of your cervical spinal column.

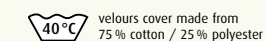
Getting used to

It usually takes two to three nights to become accustomed to the right way of sleeping. Only very few cases require a little more time.


Our cover

Please note that all our products consist of high-quality materials, which require special care, due to their special qualities. Please remove the cover carefully and without force. We recommend to hold on to the entire pillow not just one end.

Cleaning/material of cover

Cleaning of pillow core

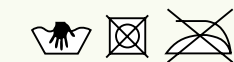After washing your pillow by hand, carefully squeeze by hand, do not wring out. To dry, place pillow on a grate or similar (place a towel underneath) and give plenty of time to dry (about a week). Do not dry near a heat source or expose to direct sunlight.

A slight discoloration of the pillow core is normal and has no impact on the special functions of the pillow.

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