SPINEFITTER

BACK IN BALANCE

SPINEFITTER® by SISSEL® Mini / Extension Kit

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EN

1. GENERAL INFORMATION

Read the following supplement to the instructions for use carefully and observe the safety instructions and warnings. The product and accessories may only be used in accordance with these instructions for use or as instructed by appropriately trained specialists. SISSEL GmbH accepts no liability for the use of the product outside the applications described in these instructions for use. The instructions for use must be kept carefully and within easy reach and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

2. PRECAUTIONARY INSTRUCTIONS

! Only use the SPINEFITTER by SISSEL Mini on backrests as described. Do not exceed the maximum application time.

⚠ Risk of injury! The Carry Strap is not a training accessory and must be removed before use. The Carry Strap is used exclusively for transporting or attaching the SPINEFITTER by SISSEL Mini to a backrest.

⚠ Risk of injury and strangulation! The SPINEFITTER by SISSEL Mini and its accessories are not toys and must be kept out of the reach of children. Do not allow children to play with it.

3. PRODUCT INFORMATION

Technical data:

Dimensions (L x W x H): Carry Strap: length approx. 170 x 2 cm

Material: Carry Strap: Polyester

Intended use

Use as a back support

Thanks to its unique design, the SPINEFITTER by SISSEL Mini attached to a backrest allows tension to be released by detaching the muscles and mobilizing the spine into an upright position. It is also possible to add the SPINEFITTER by SISSEL Trigger Tool to intensify trigger point treatment.

Indication

see enclosed instructions for use

Contra indication

· Spondylodesis

For further contraindications, see the enclosed instructions for use.

4. APPLICATION**Mounting on backrests**

! See page 19.

 Risk of injury! The SPINEFITTER by SISSEL Mini is not suitable for use on car seat backrests.

1. Before use, check the backrest of your chair or seat to ensure that it is stable and can safely support the SPINEFITTER by SISSEL Mini.
2. Place the SPINEFITTER by SISSEL Mini on the backrest so that it is level with your lower to middle back and provides the desired support.
3. Attach the SPINEFITTER by SISSEL Mini firmly to the backrest using the adjustable Carry Strap. Make sure that it is secure and tight before using it.
4. The height of the SPINEFITTER by SISSEL Mini can be adjusted using the Carry Strap. Adjust it according to your personal preferences and needs to achieve maximum comfort and support.
5. Avoid excessive use of the SPINEFITTER by SISSEL Mini and take short breaks at regular intervals to prevent overstimulation. Sitting on one side for long periods of time can lead to tension and discomfort. Lean on the SPINEFITTER by SISSEL Mini for 6-8 minutes, then change the height of the SPINEFITTER by SISSEL Mini to reach other areas of the spine and gradually treat the thoracic and lumbar spine. If you only want to mobilize a partial area, pause after 6-8 minutes so as not to overstimulate the structures. When used in combination with the SPINEFITTER by SISSEL Trigger Tool, the application time is reduced to 30 seconds to a maximum of 2 minutes.
6. Use the SPINEFITTER by SISSEL Mini as a supplement to a healthy sitting posture, but also vary your sitting position and movements to reduce the strain on your back. Stand up occasionally, stretch or take a few steps to loosen your muscles and relieve tension.

7. WARRANTY

The warranty is governed by the statutory provisions. Any warranty is excluded if goods delivered by SISSEL GmbH have been handled, treated or modified by others without consent or if the instructions for use have not been followed. Users of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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SISSEL GmbH · Bruchstrasse 48